

PTL-SDN/PTL-CD/PTL-CR



Please read this Operation Manual carefully before use, and file for future reference.

MADE IN JAPAN
OM-M0020E 002



⚠ CAUTION

- When operating the handpiece, always consider the safety of patient.
- This handpiece is designed only for dental clinic use.
- Check instrument and accessories before use.
- Should the handpiece function abnormally, cease operation immediately and return the handpiece to the dealer for repair.
- Depressing the push-button while handpiece bur is in rotation will result in **OVERHEATING** of the handpiece head. Special caution must be exercised during use to keep cheek tissue **AWAY** from the push-button of the handpiece. Contact with cheek tissue may cause the push-button to depress and burn injury to the patient may occur.
- Always use clean burs. Uncleaned burs may cause unwanted amount of pressure on the chuck.
- Do not attempt to disassemble the handpiece nor tamper with the mechanism.
- Do not drop the handpiece.

1. Mounting the air motor

If PHATELUS or MACH-LITE handpiece is now in use with NSK swivel coupling, the air motor will be ready for use only when the air turbine has been replaced with a PHATELUS motor at the coupling joint (Fig.1).

The same supply air pressure as for the air turbine can be used. To remove the air motor from the coupling, pull back the connector ring toward the hose, in the same way as for the turbine (Fig.2).

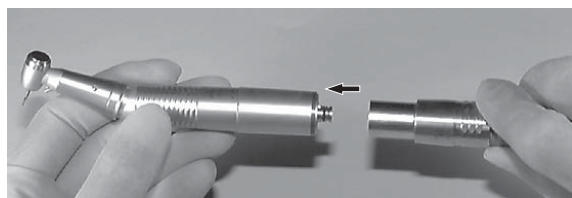


Fig.1

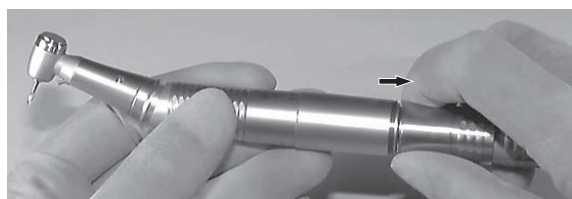


Fig.2

⚠ CAUTION

Do not pull back the connector ring when the air pressure is applied, as the motor will be ejected by the air pressure.

2. Installation and Removal of the head

To install the head, align the fiber optic pipes with the corresponding slots of the head and insert the head straight into the motor (Fig.3).

To remove the head, pull the head mounting ring toward the hose and pull the head straight out (Fig.4).



Fig.3

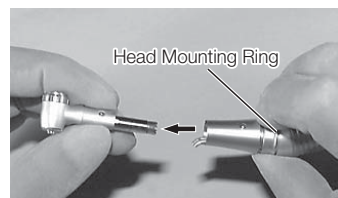


Fig.4

⚠ CAUTION

When the head is removed, take care not to bend the exposed fiber optic pipes. Keep the head installed even when the air motor is not in use.

3. Changing the direction of rotation of the air motor

The air motor will rotate in the normal direction when the marked screw on the joint cover is turned to the "F" position, and in the reverse direction when it is turned to the "R" position. Rotation of the air motor can be stopped by positioning the screw to the middle position between "F" and "R" (Fig.5).

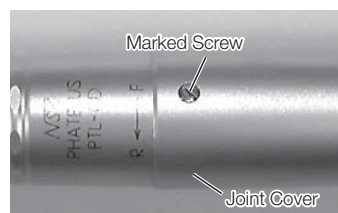


Fig.5

4. Mounting and removal of the bur

When mounting a bur, be sure to mount the head onto the motor first.

(1) NPB-PTL and NPBG-PTL

Insert a latch bur into the head and gently turn the bur to find a position at which its "D" shaped section is aligned with its counterpart in the bur tube. Insert it further into the head (Fig.6).

When the bur is further inserted in that aligned position, it will lock with a clicking sound.

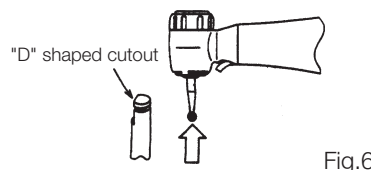


Fig.6

⚠ CAUTION

Before using the handpiece, pull out the bur gently to make sure that the bur is securely seated and locked (Fig.7).

To remove the bur, first make sure that the motor is to stop completely. Push the push button to release the bur (Fig.8).

(2) NFB-PTL and NFBG-PTL

Insert a FG bur into the head to a depth of 2mm to 3mm, then push the push button with the tip of your thumb and insert the bur into the chuck. After mounting the bur, be sure to push and pull the bur to make sure it is completely installed (Fig.9).

Pushing and pulling of the bur increases its grip force. To remove the bur, push the push button with the tip of your thumb. Remove the bur (Fig.10).

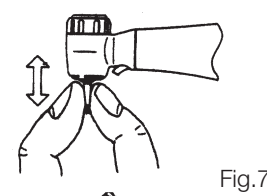


Fig.7

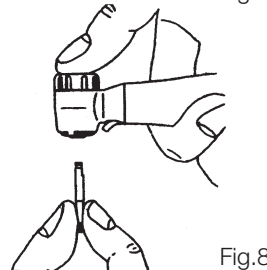


Fig.8

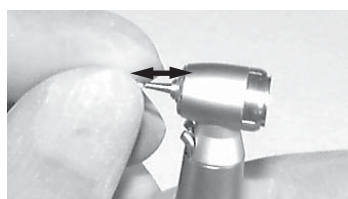


Fig.9

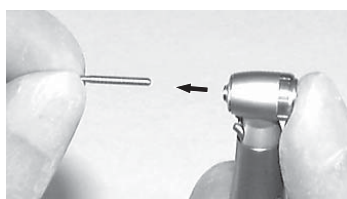


Fig.10

(3) ARS-PTL

Mount a rubber cup or the brush by screwing into the head.
To remove the rubber cup or the brush, it may be easily removed by running the air motor at a very low speed in the reverse direction while holding the cup or the brush firmly by fingers (Fig.11).

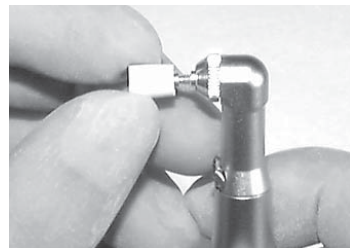


Fig.11

(4) NRS-PTL, TE-PTL, and NM-PTL

To mount the bur, open the latch blade, insert the bur by aligning the D-section of the bur with its mating hole in the bur tube. Swing back the latch blade to the original position.
To remove the bur, swing open the latch blade and pull out the bur.

(5) EVA-PTL

This can be mounted only by pushing the EVA-Tip stem into the head plunger.
For removal, push the EVA-Chip out by pressing it with the pin section of the wrench of an accessory from the rear of the head.

(6) VM-PTL

Push the push button to open the chuck. When the chuck is open, insert the hand file grip into the chuck as shown and release the push button (Fig.12).
Before using the endo instrument, lightly pull out to make sure it is securely held.
The instrument grip that can be fit into the VM-PTL is 3.8mm to 4.1mm in diameter.
To remove the instrument, push the push button and pull out the instrument.

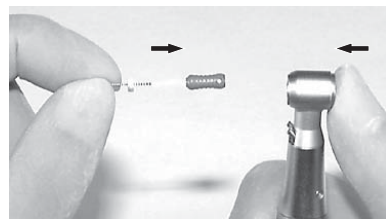


Fig.12

5. Lubrication

Always lubricate the air motor after use, and before autoclaving, with PANA- SPRAY Plus / PANA- SPRAY. Push PHATELUS Spray nozzle attachment over the PANA SPRAY Plus / PANA- SPRAY nozzle until it firmly seats. Insert the PHATELUS Spray nozzle into the rear of the air motor and spray for 2 seconds (Fig.13).

Lubrication of Contra Head.

Attach the contra head spray adapter to the end of PHATELUS Spray nozzle.
Insert the contra head drive gear into the spray adapter and spray for 2 seconds. (Fig.14)



Fig.13

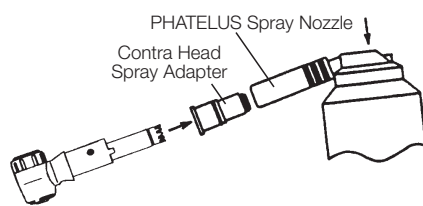


Fig.14

⚠ CAUTION

- Hold the air motor or contra head firmly when lubricating. Lubricant propellant pressure could eject the air motor or contra head from the spray nozzle. Cover the head or the air motor front with paper tissue.
- Hold the spray can upright.

6. Cleaning of the fiber optic end (Except PTL-SDN)

In case the light has become dim, clean the fiber optic end with an alcohol-soaked cotton swab (Fig.15).

Do not use a pointed tool or sharp edges to clean the fiber end. They may damage the mirror finish and reduce the light transmission efficiency.
If the fiber optic end has become damaged, send the handpiece to your local distributor for repair.

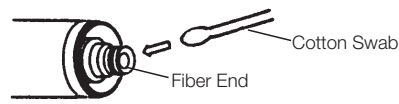


Fig.15

7. Sterilization

Steam autoclave is recommended.

Sterilization is required first time you use and after each patient as noted below.

■ Autoclave Procedure :

- ① Scrub dirt and debris from the air motor body, and wipe clean with alcohol- immersed cotton swab or cloth. Do not use a wire brush.
- ② Lubricate with PANA-SPRAY Plus / PANA-SPRAY. Refer to 5 Lubrication.
- ③ Insert into an autoclave pouch. Seal the pouch.
- ④ Autoclavable up to max. 135°C.
ex.) Autoclave for 20 min. at 121°C, or 15 min. at 132°C.
- ⑤ Keep the handpiece in the autoclave pouch to keep it clean until you use it.

* Sterilization at 121°C for more than 15 minutes is recommended by EN13060 or EN ISO17665-1.

⚠ CAUTION

Do not wipe with, or clean or immerse in, high acid water or sterilizing solutions.

8. Specification

Motor Type		PTL-SDN	PTL-CD	PTL-CR
Drive Air Pressure	MPa (kgf/cm ²)	0.25 (2.5)	0.25 (2.5)	0.25 (2.5)
Air Consumption	NL/min	32	32	32
Torque	gf-cm	190	190	540
Speed	min ⁻¹ (rpm)	25,000	25,000	6,500
Weight	g	90	81	89

9. Repair Service

For genuine NSK service and parts, always return the handpiece to your NSK dealer.

CE The EU directive 93/42/EEC was applied in the design and production of this medical device.

NAKANISHI INC.
www.nsk-inc.com
700 Shimohinata Kanuma-shi
Tochigi 322-8666,
Japan

NSK Europe GmbH
www.nsk-europe.de
Elly-Beinhorn-Strasse 8
65760 Eschborn,
Germany

NSK United Kingdom Ltd
www.nsk-uk.com
Office 5, Gateway1000,
Arlington Business Park, Whittle Way,
Stevenage, SG1 2FP, UK

NSK America Corp
www.nsk-inc.com
1800 Global Parkway
Hoffman Estates, IL 60192,
USA

NSK France SAS
www.nsk.fr
19 avenue de Villiers
75017 Paris,
France

NSK Dental Spain SA
www.nsk-spain.es
C/ Mòdena, 43 El Soho-Európolis
28232 Las Rozas, Madrid,
Spain

NSK Oceania Pty Ltd
www.nsk-inc.com
Unit 22, 198-222 Young St.
Waterloo, Sydney,
NSW 2017, Australia

NSK Middle East
www.nsk-inc.com
Room 6EA-701, 7th Floor, East Wing No.6
Dubai Airport Free Zone,
PO Box 54316 Dubai, UAE

NSK Asia
www.nsk-inc.com
1 Maritime Square,
#09-33 HarbourFront Centre,
Singapore 099253