 The EU directive 93/42/EEC was applied in the design and production of this medical device.

**NAKANISHI INC.** 

[www.nsk-inc.com](http://www.nsk-inc.com)

700 Shimohinata Kanuma-shi  
Tochigi 322-8666,  
Japan

**NSK Europe GmbH** 

[www.nsk-europe.de](http://www.nsk-europe.de)

Elly-Beinhorn-Strasse 8  
65760 Eschborn,  
Germany

**NSK United Kingdom Ltd**

[www.nsk-uk.com](http://www.nsk-uk.com)

Office 5, Gateway1000,  
Arlington Business Park, Whittle Way,  
Stevenage, SG1 2FP, UK

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**NSK Oceania Pty Ltd**

[www.nsk-inc.com](http://www.nsk-inc.com)

Unit 22, 198-222 Young St.  
Waterloo, Sydney,  
NSW 2017, Australia

**NSK Middle East**

[www.nsk-inc.com](http://www.nsk-inc.com)

Room 6EA-701, 7th Floor, East Wing No.6  
Dubai Airport Free Zone,  
PO Box 54316 Dubai, UAE

**NSK Asia**

[www.nsk-inc.com](http://www.nsk-inc.com)

1 Maritime Square,  
#09-33 HarbourFront Centre,  
Singapore 099253

Specifications are subject to change without notice.

2012.06.20 005 ©

OPERATION MANUAL

**NSK**

  0197

Please read this Operation Manual carefully  
before use, and file for future reference.

External Coolant Straight Handpiece

**Ti-Max X** **X-SG65L**  
**X-SG65**

Powerful Partners®

**MADE IN JAPAN**

 001

## Intended Use

This handpiece is designed for clinical dental use and implant surgery.

### CAUTION

- When operating the handpiece always consider the safety of the patient.
- Do not attempt to disassemble the handpiece nor tamper with the mechanism.
- Check the vibration, noise and overheating outside the patient's oral cavity before use. If any abnormalities are found, stop using the handpiece immediately and contact your dealer.
- Should the handpiece function abnormally, cease operation immediately and return the handpiece to the dealer for repair.
- Do not allow any impact on the sheath. Do not drop the handpiece.
- Always keep bur shanks clean. Dirt or debris in a chuck could cause poor bur concentricity or low chuck retention force.

## 1. Connection / Disconnecting the Handpiece and Motor

### (1) Connecting


Push the handpiece on to the motor taking care to keep it in alignment.

**X-SG65L Only**

Twist the handpiece until it locks into position with a clicking sound.

### (2) Disconnecting

Hold the motor and the handpiece, and pull it out.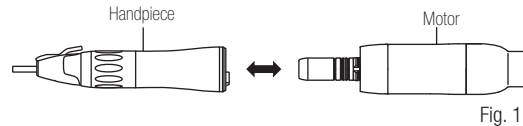

## 2. Mounting and Removing Bur


### (1) Handpiece Bur

Inserting the Bur

1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.

2) Insert the bur.

3) Turn the Bur Lock Ring in the direction of the mark [C] until it clicks. Make sure Black Dots align (Fig. 2).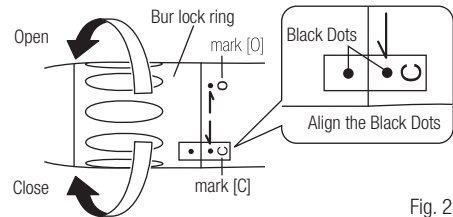

### Removing the Bur

Turn the Bur Lock Ring in the direction of mark [O] until it clicks and remove the bur.

### (2) Contra-angle bur

Inserting the Bur

1) Turn the bur lock ring in the direction of mark [O] until it clicks.

2) Insert the supplied Bur Stopper into the chuck. (Fig. 3)

3) Insert the Contra-Angle Bur until it stopped.

4) The bur will be chucked by turning the bur lock ring in the direction of mark [C] until it clicks and Black Dots line up.



### Removing the Bur

Turn the Bur Lock Ring in the direction of mark [O] until it clicks and remove the bur.

### ■ Remove the Bur Stopper

1) Turn the Bur Lock Ring in the direction of mark [O] until it clicks.

2) Insert the supplied Bur Stopper puller into the chuck. Pull out the Bur Stopper by hooking at the slit of Bur Stopper. (Fig. 4)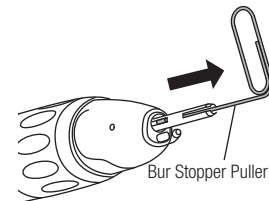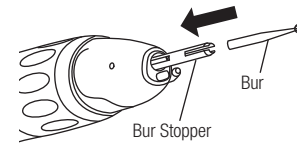

## CAUTION for Bur Usage

### Heat Generation Problem

- Bur position is improper or it is not mounted.  
--> Mount the bur correctly.
- During operation, Bur Lock Ring had been turned.  
--> Do not turn the Bur Lock Ring while the handpiece is rotating.

### Bur Jump Out Problem

- Incorrect mount of the bur/bur stopper.  
--> Mount the bur/bur stopper firmly.
- During operation, Bur Lock Ring had been turned.  
--> Do not turn the Bur Lock Ring while the handpiece is rotating.

- Mount short shank contra-angle bur.  
--> Do not mount this bur.

### Bur Breakage Problem

- Mount damage or sub-standard burs.  
--> Do not mount those burs.
- Exceed drive motor speed or manufacture's recommended bur speed.  
--> Do not exceed drive motor speed or manufacture's recommended bur speed.

## 3. External Irrigation Nozzle

Connect the irrigation tube to the external irrigation nozzle firmly.

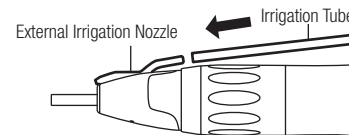
Fig. 5

## 4. Maintenance

### CAUTION

After each patient, be sure to keep the handpiece after cleaning, lubrication, and sterilization. If the blood is left on the handpiece, it can be clotted causing rusty.

Refer to the Maintenance Guide **OM-Z0135E**.

This handpieces can be autoclavable up to Max.135°C.

This handpieces can be cleaned and disinfected with a Thermo-Disinfector.

\*Cleaning

Clean the external irrigation nozzle inside with clean water.

## 5. Cleaning the Glass Rod Optic

Clean the Glass Rod Optic with cotton swab soaked in alcohol.

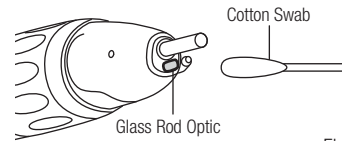
Fig. 6

### CAUTION

Do not use a sharp tool to clean the glass rod optic. It could damage the glass rod optic and decrease the light transmission. If it becomes dim, contact your dealer.

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**ENGLISH** Specification

Model	X-SG65L	X-SG65
Gear Ratio	1:1	
Allowable Max Speed	40,000 min <sup>-1</sup> (rpm)	
Bur	ø2.35mm Handpiece Bur and ø2.35mm CA Bur	
Bur applied Length	29.4mm (Handpiece Bur)	
Max Bur Length	44.5mm (Handpiece Bur)	
Max Bur diameter	Ø4.0	
Irrigation	External Irrigation	
Optic	Glass Rod	-
Use Environment	Temperature : 0 – 40 °C, Humidity : 30 - 75% HR, Atmospheric Pressure : 700 - 1,060hPa	
Transportation and Storage Environment	Temperature : -10 – 50 °C, Humidity : 10 - 85% RH, Atmospheric Pressure : 500 - 1,060hPa	

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